



NEW:
Results within a maximum of 8 minutes

DrugWipe® 5 S sets standards

The new saliva test DrugWipe® 5 S was used in traffic controls by the police in Rhineland-Palatinate. All positive results were verified by means of blood analysis.

In a large scale operation, the police forces extensively used DrugWipe® as well as other drug tests in comparison for several days. The data set forth herein were collected during a festival in Germany, Nature One 2011.

During this event, screening with DrugWipe® was able to detect drug consumption of numerous visitors.

DrugWipe® 5 S – sensitive & reliable.

Detection of THC in blood as low as 1 ng/ml


- ✓ Saliva test DrugWipe® 5 S was used in the field test
- ✓ A total of 99 drivers were tested on-site
- ✓ 392 out of 396 test results were correct
- ✓ All 16 THC-pretests were verified by blood analysis
- ✓ Δ9-THC of 0,7-38 ng/ml was detected in the blood

		THC	AMP/ MET	COC	OPI
Sensitivity	[%]	94	100	100	100
Specificity	[%]	100	99	98	100
Accuracy	[%]	98	99	98	100

www.securetec.net/Drugwipe

Performance data DrugWipe® 5 S in comparison

Consistency DrugWipe® / blood analysis

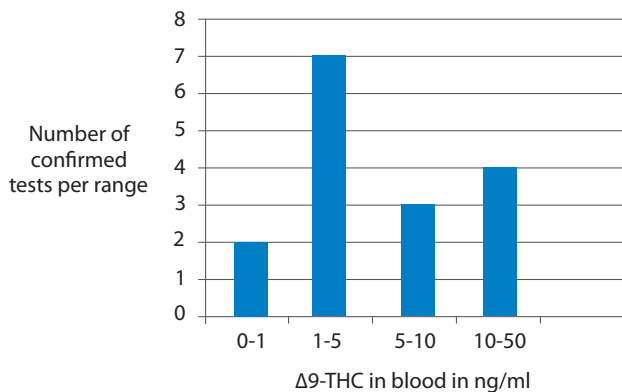

Features which make DrugWipe® 5 S outstanding:

- Capability of DrugWipe® 5 S to recognize traces of THC has been significantly improved. The limit of detection in saliva is 10 ng/ml.
- Even minimal concentrations are reliably detected, down to the range of 1 ng THC/ml in blood.
- The test lines are more pronounced and thus can be easily identified in the field.

Comparison of DrugWipe® 5+ with the new DrugWipe® 5 S

In the course of the EU project DRUID, 135 DrugWipe® 5+ tests were evaluated. In a comparison of eight saliva tests, DrugWipe® 5+ showed the best characteristic values. Further improvement of the already very good performance could be achieved by the successor product DrugWipe® 5 S. Sensitivity and reliability of all parameters are now exceeding 95% on average.

		DrugWipe® 5+	DrugWipe® 5 S
Sensitivity	[%]	91	96
Specificity	[%]	96	99
Accuracy	[%]	93	99

See it for yourself – Let's test DrugWipe® 5 S together



Securetec Detektions-Systeme AG
Eugen-Sänger-Ring 1
85649 Brunenthal Germany
T +49 89 203080-1651
F +49 89 203080-1652
info@securetec.net
www.securetec.net

© 2012 Securetec Detektions-Systeme AG